

SafeKare™ Base Stand



The SafeKare Base Stand is an optional part of the SafeKare Monitoring System.

The Base Stand serves as a vertical stand and battery charger for the Caregiver Receiver Unit. It provides the caregiver a location to place the receiver for easy access and charging, while still providing client monitoring.


Operation Summary

The caregiver plugs the base stand into the wall power adapter.


The caregiver may then place the

Caregiver Receiver Unit (CRU) into the Base Stand. The CRU may rest in the stand powered on or off and will charge the internal battery pack in either condition (similar to a cordless phone). If powered on, the CRU continues to monitor the client tag location and will alert the caregiver if the client wanders from the monitored area. The Base Stand includes an additional alarm speaker that provides a louder alarm than the CRU alone. The CRU may be removed from and placed into the base stand at any time, without disconnecting power from the base stand.

Client Inside Control Area

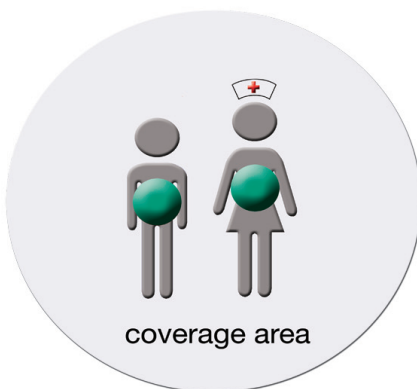

Client Outside Control Area Alarm Sounds

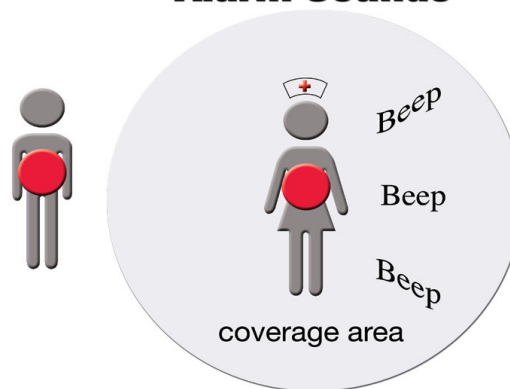

SafeKare™ Base Stand

Base Stand

Power Supply	9 -12 volt DC
Dimensions	6" x 4" x 2.5"
Weight	14 ounces
Display	LED illuminated when charging
Sealing	None
Temperature	
Operating	0° to +70° C (+32° to +158° F)
Storage (non-operating)	-10° to +85° C (+14° to +185° F)

SafeKare™ Base Stand
Model Number: BS101A

Related Products

- Caregiver Receiver Unit
- Wall Charger
- Car Power Adapter

Microwave Monolithics Incorporated
2263 Ward Avenue
Simi Valley, CA 93065
(805) 584-6642

Website: www.micro-mono.com/safekare
Email: sales@micro-mono.com

SafeKare™ is a registered trademark.