

# SafeKare™ Caregiver Receiver



## The SafeKare Caregiver Receiver is a part of the SafeKare Monitoring System.

The Caregiver Receiver Unit (CRU) is carried by the caregiver and alerts the caregiver if the client moves out of the monitored area. The monitored area is selected by a slide switch on the side of the CRU. The range of the monitored areas are: short (15-25 feet), medium (25-50 feet), and long (50-100+ feet).


### Operation Summary

The caregiver turns on the CRU and commands the unit to learn the client tag identification. This learning procedure allows a single CRU the capability of monitoring different client tags (only one at a time). This allows the caregiver to select a unique tag type for the client to wear. If the client always wears the same tag, then this learning procedure is only required once. The caregiver then selects the appropriate range setting for the monitored area. The range setting will vary depending on the type of area monitored (example: home, store, mall, park, etc.). The SafeKare system is now ready for use.

### Client Inside Control Area



### Client Outside Control Area Alarm Sounds

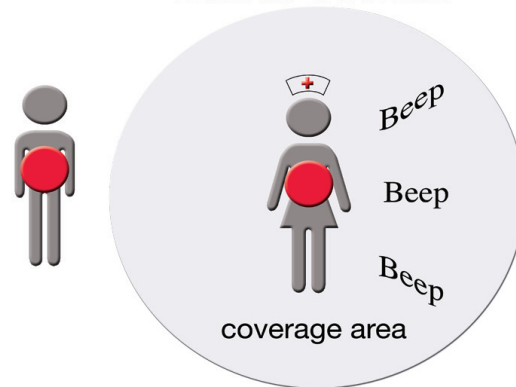

The CRU will remain operational, monitoring the signal from the client tag, until turned off. The client tag emits its unique ID number every 10 seconds. If the client wanders from the monitored area, the CRU will sound an audible alarm and blink the alarm LED indicator. On the first missed ID, the CRU beeps the alarm once. Upon the second consecutive missed ID, the CRU continually beeps the alarm until manually reset by the caregiver or until the client re-enters the monitored area.

# SafeKare™ Caregiver Receiver

## Caregiver Receiver Specifications

Dimensions	
Excluding antenna	2-5/8" x 2-1/2" x 7/8" without belt clip
Including antenna	4-1/4" x 2-1/2" x 7/8" without belt clip
Weight	2.75 ounces (78.5 grams) with belt clip
Battery Voltage	4.2V ±15%
Battery Type	3-cell NiMH (not user replaceable)
Battery Charging Voltage	+9 to +14 VDC
Average Battery Life (between recharges)	12 hours
Sealing	Dust resistant
Humidity	98% non-condensing
Temperature	
Operating	0° to +70° C      (+32° to +158° F)
Storage (non-operating)	-10° to +85° C      (+14° to +185° F)

SafeKare™ Caregiver Receiver Unit  
Model Number: CRU101A

### Accessories

- Belt Clip
- Wall Charger

### Related Products

- Patient Monitoring Tags
  - Hanging FOB
  - Wrist/ankle FOB
  - ID Badge
- Base Stand
- Car Power Adapter
- Earphone

Microwave Monolithics Incorporated  
2263 Ward Avenue  
Simi Valley, CA 93065  
(805) 584-6642

Website: [www.micro-mono.com/safekare](http://www.micro-mono.com/safekare)  
Email: [sales@micro-mono.com](mailto:sales@micro-mono.com)

SafeKare™ is a registered trademark.